

ELEVISION™ 4K LAPAROSCOPE

DEVICE USE TROUBLESHOOTING ACTIVITY HANDOUT

Review the Laparoscope IFU for more information on troubleshooting potential problems.

INSTRUCTIONS	TOPICS
<ul style="list-style-type: none">• Read the potential problems that clinicians may experience while using the device.	1. Blurry, cloudy, or dark image
<ul style="list-style-type: none">• Provide a solution to each potential problem.	2. Poor image quality
<ul style="list-style-type: none">• Refer to the Instructions for Use (IFUs) for the laparoscope.	

1. Potential Problem: Blurry, cloudy, or dark image

Solution

- Check for moisture on the proximal end.
- Check to see if the glass surfaces on the laparoscope are soiled.
- Clean and dry the laparoscope manually, then reprocess and check the water quality.
- Check for leaks in the lens system and return the laparoscope for repair or replace if needed.
- Check to see if the lens system is faulty and return the laparoscope for repair or replace if needed.

2. Potential Problem: Poor image quality

Solution

- Check for contaminants on the irradiation surface of the illumination fibres which can burn in during use.
- Inspect distal and proximal ends of the fibres for contamination.
- Ensure that the irradiation surface of the illumination fibre is clean.
- Return the laparoscope for repair if a problem is observed or suspected.